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INSTRUCTIONS FOR USE

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Pulsossimetro Parlante.

Talking Pulse Oximeter.

Oxímetro De Pulso Parlante.

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Sprechendes Pulsoximeter.

iVoice
Oxy

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Thanks for choosing VIBAS.

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ENGLISH INSTRUCTIONS FOR USE

USE AND COMMISSIONING

Insert the batteries, paying attention to the "plus" and "minus" signs.

To turn on the pulse oximeter, press the power button.

To start the measurement, insert your finger into the device without moving it.

The pulse oximeter turns off automatically after 8 seconds of inactivity.

SETTINGS MENU:

1: ADJUST VOLUME OR MUTE

Press the power button for two seconds to enter the settings menu; the display and voice indicate that we are in the *volume control* menu.

Press the power button once and a voice will announce the selected volume; continue until the desired volume is reached.

2: CHANGE THE DISPLAY ORIENTATION

Press the power button for two seconds to switch from the *volume control* menu to the *display orientation* menu.

Press the power button once and the display will change orientation. Continue until the desired orientation is reached.

3: CHANGE THE LANGUAGE

Press the power button for two seconds to switch from the *display orientation* menu to the *language change* menu.

Press the power button once and a voice will announce the chosen language. Continue until the desired language is found.

4: EXIT THE SETTINGS MENU

Press the power button for two seconds to exit the menu and save your settings.

The settings menu still exits automatically after 8 seconds of inactivity.

REPEATING THE MEASUREMENT

By leaving your finger in the device, the measurement will be repeated several times.

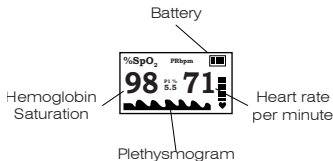
Press the power button to repeat the measurement.

BATTERY REPLACEMENT NOTIFICATION

When the batteries need replacing, a voice alert indicates the low battery status.

HOW TO READ THE DISPLAY

After switching on, the display shows the following:

PACKAGE CONTENTS

A talking pulse oximeter;
Two 1.5 V AAA batteries;
An Instruction Manual;

TECHNICAL DATA

Model: iVoice Oxy 1 (PO-A2AO)

Dimensions: 58 x 35 x 35 mm

Measurement range (SpO₂): 70% ÷ 99% (±1%)

Measurement range (PR): 30 bpm ÷ 240 bpm (±1bpm)

Two 1.5 V AAA batteries

Material: plastic, metal

Protection class: IP22

WARNINGS, CLEANING AND CARE

Clean the appliance before use.

Clean the pulse oximeter with a damp cloth, but do not allow water to penetrate inside.

For cleaning, do not use chemical detergents.

Store the pulse oximeter in environments with temperatures between -10°C and +40°C and humidity between 10% and 80%.

ATTENTION

- Never leave children unattended with packaging materials or the pulse oximeter.
- Do not use the pulse oximeter if it is damp or wet.
- Remove the batteries if the pulse oximeter will not be used for an extended period of time.
- Do not expose the pulse oximeter to heat sources.
- Do not recharge batteries or throw them into a fire, risk of explosion.
- If fluid leaks from the batteries, avoid contact with skin, eyes, and mucous membranes.

NOTES ON USE

- Nail cosmetics or false nails can cause incorrect measurements.
- The device arrives already calibrated and ready to use.
- Do not use if there are intravenous or arterial cannulas or blood pressure monitors in use on the limb.

WARRANTY

The product is covered by a 2-year warranty against manufacturing defects.

The warranty expires in the following cases:

- Damage caused by improper use.
- For components subject to wear.
- Customer responsibility.
- If the device has been opened.

To claim the warranty within its validity period, the customer must provide proof of purchase of the product.

You can assert your warranty rights against VIBAS within a period of two years.

For any complaints, please contact our customer service at the following numbers:

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